

BloodSTOP[®]

CONTROLS BLEEDING FAST

BloodSTOP Instructions for Use

BloodSTOP Hemostatic Gauze is made of regenerated, oxidized and etherized natural fiber cellulose, which is water soluble and biocompatible. When BloodSTOP material comes into contact with blood and exudate, it turns into a clear gel that adheres to the wounded area and seals the wound.

Indications For Use:

Emergency and therapeutic use in the control of bleeding from the skin and other surface wounds where temporary control of bleeding is required.

Directions

1. Apply BloodSTOP to the cleaned wound. Use additional layers as needed.
2. If necessary, place a piece of wet cotton gauze over the BloodSTOP material and apply pressure.
3. When the bleeding stops, keep BloodSTOP gauze moist with sterile water or saline.
4. Combine with standard wound dressings for further protection if desired.
5. To remove excess BloodSTOP material, rinse with water or saline and wipe gently if necessary.
6. Should the bleeding recur, repeat the above steps.

Nose Bleeding

1. Cut BloodSTOP into a 0.5" x 2" strip. Use dry forceps to pack BloodSTOP strip into the bleeding nostril. Apply pressure as needed.
2. After the bleeding stops completely, gently rinse off with water.

Trauma (Venous Bleeding)

1. Apply one or more layers of BloodSTOP to the wound.
2. Place a piece of wet gauze over the BloodSTOP material and apply pressure.
3. When the bleeding stops, keep BloodSTOP moist with sterile water or saline.
4. BloodSTOP can be rinsed away with water.

Dialysis

1. Apply one or more layers of BloodSTOP gauze to the wound puncture.
2. Place a piece of wet gauze over the BloodSTOP material and apply pressure.
3. Keep BloodSTOP moist with sterile water or saline as needed.
4. After bleeding stops completely, BloodSTOP can be rinsed away with water.

WARNING: In case of swelling or redness, discontinue use of BloodSTOP.

STORAGE: Keep BloodSTOP sealed packets in a cool, dry place for maximum shelf-life.

INGREDIENTS: Regenerated, etherized and oxidized natural fiber cellulose.



NETWORK MEDICAL PRODUCTS, LTD
Ripon, N. Yorkshire HG4 1NS, UK



Symbols Used on Labeling



Single use



Use until year and month



Batch number



See instructions for use



Company Address



Do not use if package is damaged



Sterile unless package is opened or damaged.
Method of Sterilization: Irradiation



CE Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC as amended by 2007/47/EEC

LifeScience PLUS, Inc.

P.O. Box 60783, Palo Alto, CA 94306

Toll Free: 1-877-587-5433 - U.S.A.

Tel: +1-650-565-8172 - International

Fax: 1-650-336-1130

www.LifeSciencePlus.com

www.blood-stop.com

Email: info@lifescienceplus.com

