


# **BloodSTOP**<sup>®</sup>

## **CONTROLS BLEEDING FAST**

BloodSTOP<sup>®</sup> is a water soluble gauze which stops bleeding in less than a minute. Upon contact with blood, it speeds coagulation and expands into a transparent gel that seals the wound and enables easy wound monitoring.

### **HOW TO USE BloodSTOP**

1. Clean the wound.
2. Apply BloodSTOP on the bleeding wound (BloodSTOP may be cut to fit a specific wound).
3. Apply light pressure until bleeding stops completely, usually less than one minute.
4. If desired, the BloodSTOP material surrounding the wound may be moistened with water so it adheres to the skin.
5. Once bleeding has stopped completely, rinse off excess BloodSTOP by saturating with water and gently wiping away the gel, being careful not to disturb the wound itself.
6. If bleeding has not stopped after the first application of BloodSTOP, you may apply more, following the instructions above. If bleeding continues, contact a Doctor or Emergency Room.

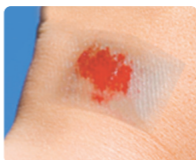

### **HOW TO USE BloodSTOP for Nosebleeds**

1. Cut BloodSTOP into a small strip.
2. Pack the BloodSTOP strip into the nostril, as close to the source of bleeding as possible.
3. Compress the bleeding nostril to assist in stopping the blood flow.
4. If desired, gently rinse off the remaining BloodSTOP with water once bleeding has stopped.
5. If bleeding has not stopped after the first application of BloodSTOP, you may apply more, following the instructions above. If bleeding continues, contact a Doctor or Emergency Room.

## FOR MEDICAL EMERGENCIES, CONTACT A DOCTOR

**INGREDIENTS:** Regenerated etherized and oxidized natural fiber cellulose.

**WARNING:** In case of wound swelling or redness for more than 48 hours after use of BloodSTOP®, see a Doctor. In case of emergency, see a Doctor.  
For external use only.

**CAUTION:** BloodSTOP is not to be used on dirty, contaminated or infected wounds. Consult your doctor if you notice any itching, burning, rash or signs of infection such as fever, pain, redness or swelling.

**STORAGE:** Keep BloodSTOP sealed packets in a cool, dry place for maximum shelf life.



NETWORK MEDICAL PRODUCTS, LTD  
Ripon, N. Yorkshire HG4 1NS, UK



### Symbols Used on Labeling



Single use



Use until year and month



Batch number



See instructions for use



Company Address



Do not use if package is damaged



Sterile unless package is opened or damaged.  
Method of Sterilization: Irradiation



CE Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC as amended by 2007/47/EEC

#### LifeScience PLUS, Inc.

P.O. Box 60783, Palo Alto, CA 94306

Toll Free: 1-877-587-5433 - U.S.A.

Tel: +1-650-565-8172 - International

Fax: 1-650-336-1130

[www.LifeSciencePlus.com](http://www.LifeSciencePlus.com)

[www.blood-stop.com](http://www.blood-stop.com)

Email: [info@lifescienceplus.com](mailto:info@lifescienceplus.com)

