

PATIENT INSTRUCTIONS AFTER SHEATH REMOVAL:

A BloodSTOP product was initially applied to your wound to stop it from bleeding. It has been covered with a top dressing.

Take care not to disturb your wound. Do not rub, scratch, or pick at the wound or the dressing.

Keep the wound dressing clean and dry.

Follow your doctor's instruction on how long to keep BloodSTOP on your wound, but in no case should you leave it on for more than 24 hours after returning home.

Removal steps:

1. Carefully peel back enough of the top dressing to expose the BloodSTOP material.
2. Saturate the BloodSTOP material with single-use container, sterile normal saline until it becomes a soft gel.
3. Carefully remove the top dressing.
4. Gently wipe or wash away the saturated gel, being careful not to disturb the wound.
5. Let it all dry.

*Note: Removal steps may be followed in the shower.*

Keep the wound protected from injury during the remainder of the healing period.

You may apply a new dry dressing over the wound if desired.

Follow all hospital discharge instructions for the observation of your wound carefully.