

BloodSTOP[®]

CONTROLS BLEEDING FAST

BloodSTOP Instructions for Use

BloodSTOP Hemostatic Gauze is made of etherified, oxidized, regenerated natural fiber cellulose, which is water soluble and biocompatible. When BloodSTOP material comes into contact with blood and exudate, it turns into a clear gel that adheres to the wounded area and seals the wound.

Indications For Use:

Emergency and therapeutic use in the control of bleeding from the skin and other surface wounds where temporary control of bleeding is required.

Directions

1. Fold BloodSTOP in half to make two layers. Apply to the cleaned wound with dry instruments or gloves.
2. Apply pressure on the wound site.
3. Use additional layers of BloodSTOP as needed. Moisten with saline if necessary for adherence to the skin surface.
4. Combine with standard wound dressings for further protection if desired.
5. To remove excess BloodSTOP material, rinse with water or saline and wipe gently if necessary.
6. Should the bleeding recur, repeat the above steps.

Nose Bleeding

1. Cut BloodSTOP into a 0.5" x 2" strip. Use dry forceps to pack BloodSTOP strip into the bleeding nostril, as close to the broken vessel as possible. Apply pressure.
2. After the bleeding stops completely, gently rinse off with water.

WARNING: Do not use for dental extractions. In case of swelling or redness, discontinue use. For external use only. COMPLETE INSTRUCTIONS ENCLOSED.

STORAGE: Keep BloodSTOP in a cool dry place for maximum shelf life

INGREDIENTS: Regenerated, etherified and oxidized natural cellulose.



Welkang Ltd
Suite B, 29 Harley Street
LONDON, W1G 9QR, U.K.



Symbols Used on Labeling



Single use



Use until year and month



Batch number



See instructions for use



Company Address



Do not use if package is damaged



Sterile unless package is opened or damaged.
Method of Sterilization: Irradiation



CE Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC as amended by 2007/47/EEC



LifeScience PLUS, Inc.

2520A Wyandotte St., Mountain View, CA 94043, U.S.A.

P.O. Box 60783, Palo Alto, CA 94306, U.S.A.

Toll Free: 1-877-587-5433 – U.S.A.

Tel: +1-650-565-8172 – International

Fax: 1-650-336-1130

www.LifeSciencePlus.com

www.blood-stop.com

Email: info@lifescienceplus.com

