

BloodSTOP[®]

CONTROLS BLEEDING FAST

BloodSTOP[®] Instructions for Use

BloodSTOP[®] Hemostatic Dressing is made of 100% natural etherified sodium carboxymethyl cellulose, which is water soluble and biocompatible. When BloodSTOP[®] material comes into contact with blood and exudate, it turns into a clear gel that adheres to the wounded area and seals the wound.


Indications For Use:

For external, short term, use for the control of bleeding from lacerations, cuts, abrasions, and venipuncture, and for wound management when a moist wound environment is needed. For the general population and (EU countries only) for those on anticoagulant medications.


Directions:



1. Apply 1 or more layers to clean wound with dry or gloved hand. Apply continuous pressure until bleeding stops completely, usually 2-3 minutes. To avoid sticking to hand, roll fingers off of sticky gel.

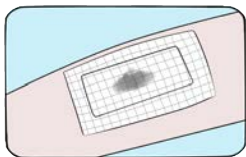

2. Cover with additional dressing to maintain moist wound environment. If bleeding continues, seek medical attention.

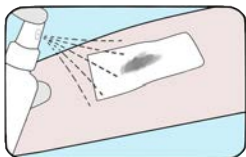

3. Change within 24 hours. To remove, saturate with sterile water or sterile saline and gently wipe away the gel being careful not to disturb the clot. Repeat above steps until wound heals.

Nose Bleeding:



1. Insert gently by hand, or with cotton swab, as close to the source of bleeding as possible, toward center and back of head.



2. Compress both nostrils from above until bleeding stops. Repeat if necessary. Seek medical attention if bleeding does not stop.

3. To remove, gently tug or squeeze gel out of the nostril. Gently rinse with sterile water or sterile saline.

WARNING: Do not use for dental extractions. In case of swelling or redness, discontinue use. For external use only. Cover BloodSTOP® with a wrap or adhesive bandage to maintain pressure if the patient is to be moved. Do not reuse, as device will no longer be sterile after initial use, and could result in infection.

STORAGE: For maximum shelf life, storage temperature of 15-30 °C (59-86 °F) is recommended.

INGREDIENTS: Purified, water soluble, etherified sodium carboxymethyl cellulose.

EC REP WellKang Ltd.
The Black Church
St. Mary's Place
Dublin 7, D07 P4AX, Ireland



Not made with natural rubber latex

Symbols Used on Labeling



Single use



Use until year and month



Reference number



See instructions for use



Batch number



Do not use if package is damaged



Company address



Do not resterilize



Temperature Limits



Sterile unless package is opened or damaged.
Method of sterilization: irradiation



CE Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC as amended by 2007/47/EEC.



LifeSciencePLUS, Inc.

2520A Wyandotte Street, Mountain View, CA 94043, USA

Mailing: P.O. Box 60783, Palo Alto, CA 94306

1.877.587.5433 (U.S.) 1.650.565.8172 (International)

www.lifescienceplus.com; www.bloodstopix.com

© 2019 LifeScience PLUS, Inc. All Rights Reserved.

BloodSTOP® is a registered trademark. BloodSTOP® and

BloodSTOP® iX are covered by U.S. Patent No. 7,262,181

Manufactured in the USA

LifeScience PLUS
REVOLUTIONARY WOUND CARE

Rev. 2019-09